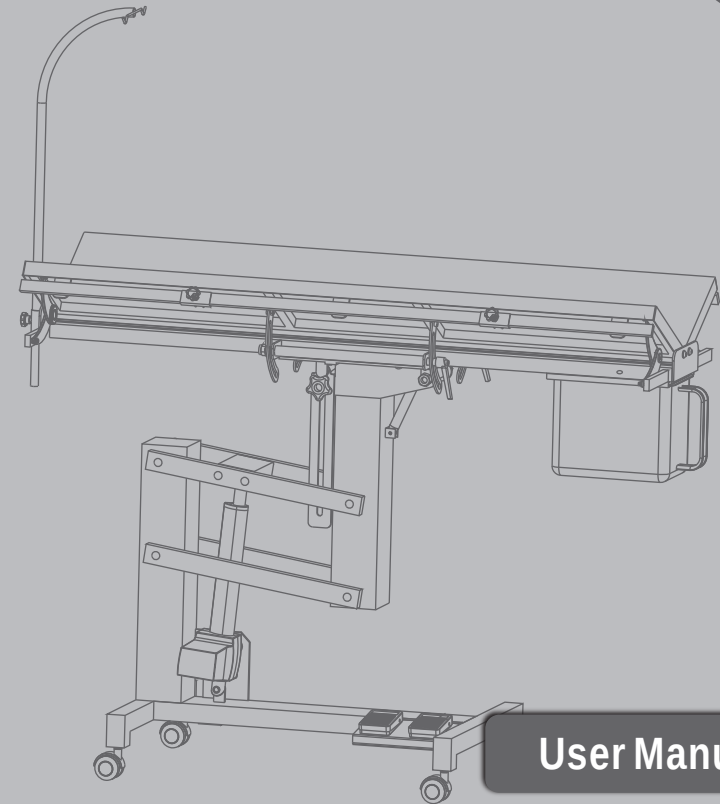
blovi

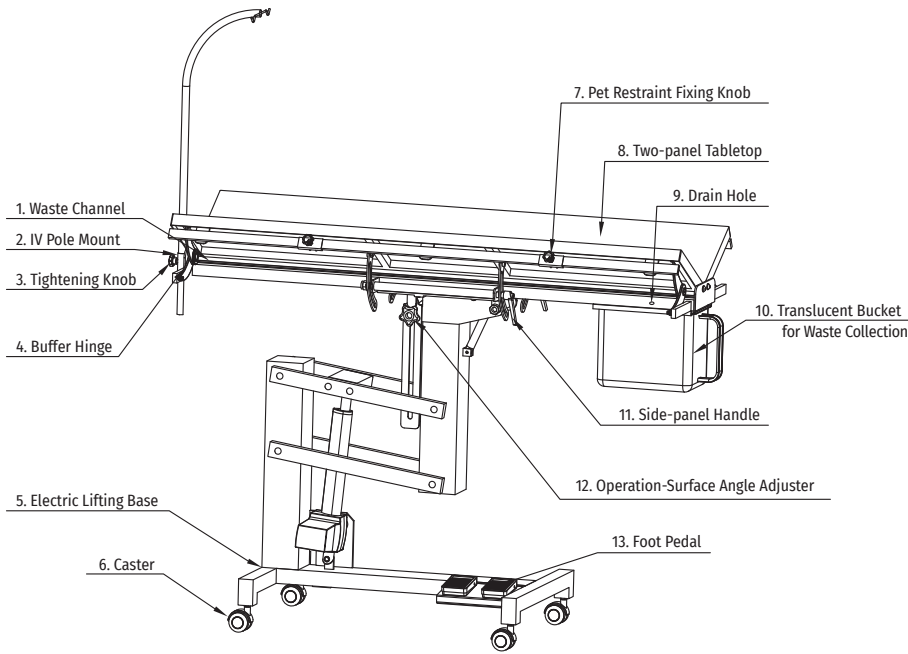
blovi

BL-VT01

New
Generation

User Manual

- | | |
|------------------------------|---|
| 1. Waste Channel | 8. Two-panel Tabletop |
| 2. IV Pole Mount | 9. Drain Hole |
| 3. Tightening Knob | 10. Translucent Bucket for Waste Collection |
| 4. Buffer Hinge | 11. Side-panel Handle |
| 5. Electric Lifting Base | 12. Operation-Surface Angle Adjuster |
| 6. Caster | 13. Foot Pedal |
| 7. Pet Restraint Fixing Knob | |

Dimension	L1400×W500×H700—1200(mm) 55-1/8×19-11/16×27-9/16×47-1/4(inch)
Package	L1460×W580×H850(mm) 57-31/64×22-53/64×33-30/64(inch)
Net Weight	80kg
Gross Weight	95kg
Loading Capacity	120kg
Volume	0.73cbm

Brief Introduction

A two panel table top system that adjusts from horizontal to near vertical to hold the pets in the desired position during surgery.

10cm (4") stainless steel trough encourages fluid run off into a drain.

Transparent drainage bucket collects waste fluid, and offers direct vision of waste liquid volume.

Powerful and durable electric lifting actuator holds 120kg (240lbs) on the surgery top (powered by Linak), provides long service life.

Water resistant foot pedal set provides easy and comfortable table height adjustments.

Tight and secured V-top and table tilting adjustments.

Ergonomically designed, each side panel is 9", based upon average forearm length.

Two sliding, quick release easy controls are provided on each side of the rail provides secure pet restraint.

Special buffer system (consisting of buffer hinges and air springs) that cushions the dropping impact of the whole top and V-top side panels.

Medical class casters move smoothly and brake easily.

Overall size of the top: 20"W× 55"L.

Table can be lifted from 28" to 47".

Worktop Swivel Angle: 0° - 45°.

Parts list

Table mainframe, 4 medical class casters, waste bucket, infusion pole, tightening knobs.

Package Inspection

The manufacturer is not liable for any damage caused by a third party "Transport or Delivery Company".

The day the unit is delivered, an inspection of the outer packing should be made for any damage to the packing box.

Then the unit should be unpacked carefully; all components, accessories and the table itself should be inspected for any damage or loss of component parts.

If any damage is evident, the carrier or dealer should be notified immediately.

Safety Warnings

1. Use only as described in this manual. Use only manufacturer's recommended attachments.
2. This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product by a person responsible for their safety.

Children should be supervised to ensure that they do not play with the product.

3. DO NOT modify, disassemble or repair.
4. DO NOT leave the appliance plugged in when not in use. Unplug from outlet when not in use and before servicing.
5. To reduce the risk of fire, or electric shock – DO NOT use outdoors or on wet surfaces. DO NOT place the product on an uneven or soft ground.
6. DO NOT use with damaged cord or plug.
7. DO NOT cut, damage, or modify the cord. DO NOT pull or carry by cord, use cord as a handle, close door on cord, or pull cord around sharp edges or corners. DO NOT pull, twist, or bend the cord with unnecessary force. DO NOT run the product over cord. Keep cord away from heated surfaces. DO NOT place heavy objects on the cord or allow the cord to get pinched in between objects. DO NOT wind the cord around the product.
8. DO NOT unplug by pulling on cord. To unplug, grasp the plug, not the cord.
9. DO NOT handle plug or the product with wet hands.
10. DO NOT use the product near flammable or combustible liquids such as gasoline or use in areas where they may be present. The fumes from these substances can create a fire hazard or explosion.
11. Make sure the local voltage and frequency of the power meet the requirements on the label of the product.
12. Avoid using the product in dusty, vibrating and wet environment and in chemical storage place.
13. Neither people nor animals are allowed to be under the tabletop especially when it is power on.
14. Gently control the table to rise or go down slowly and safely.
15. Turn off all controls before unplugging.

Assembly Instructions

1. Clockwise screw the casters into predrilled holes at the bottom of the table base.
2. Put on the waste collection box by sliding into the trough.
3. Insert the IV pole into the fixture clamp and tighten the knob when necessary.
4. Adjust the tabletop angle, whole top tilting angle and the table base height accordingly.

Operation Instructions

1. Move the table to the desired position and lock the casters.
2. Clean and disinfect the tabletop.
3. Power on and step on the "up" and "down" to adjust the height to a desired level.
4. Turn down the side-panel handle to release the panel. Adjust the angel of the panel and turn up the side-panel handle to lock the panel in position.
5. Insert the IV pole in the IV pole mount and turn the tightening knot to fix it.
6. Gently place the patient animal on the tabletop.
7. Start operation.
8. Move the patient away after the operation.
9. Clean the tabletop. Before cleaning, users can use the operation surface angel adjuster to tilt the tabletop for easier cleaning.

Maintenance

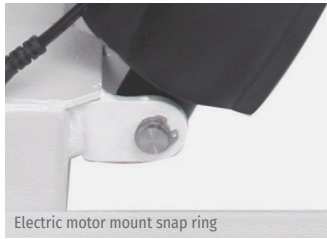
In between usages, the table should be cleaned with a soft cloth or sponge using a mild cleaning agent (and then disinfected).

Parts Replacement

Electric motor replacement


1. Disconnect the powder cord from the wall socket. Remove the tabletop from the frame assembly, and set it aside. Remove the grooming arm.
2. Disconnect the electric motor cable from the motor control unit. Remove the snap rings and the retaining bolts that hold the electric motor to the frame assembly. (See Figure 1 and 2)
3. Using help, lift he frame assembly up, and remove the entire motor assembly from the frame.
4. Install the new motor assembly in the reverse order. Reconnect the cable to the motor control unit.
5. Plug the power cord into a wall socket and test the table for proper operation.

Electric motor mount snap ring

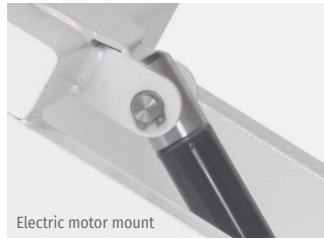Figure 1

Electric motor mount

Figure 2

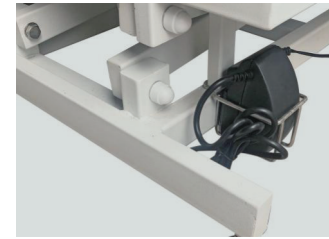
Figure 6

Control pedal replacement

1. Disconnect the powder cord from the wall socket.
2. Remove the three screws holding the control pedal to the frame assembly.
3. Disconnect the control cable connector from the motor control unit. (See Figure 3)
4. Install a new pedal in the reverse order, and re-connect the cable to the motor control unit.
5. Plug the power cord into a wall socket and test the table for proper operation.

Figure 3

4. Take the transformer off the mounting rack. (See Figure 6)
5. Install a new transformer in the reverse order.

Transformer replacement

1. Disconnect the powder cord from the wall socket.
2. Disconnect the power cable connector from the control pedal. (See Figure 4)
3. Disconnect the control cable connector from the electric motor unit. (See Figure 5)

Figure 4

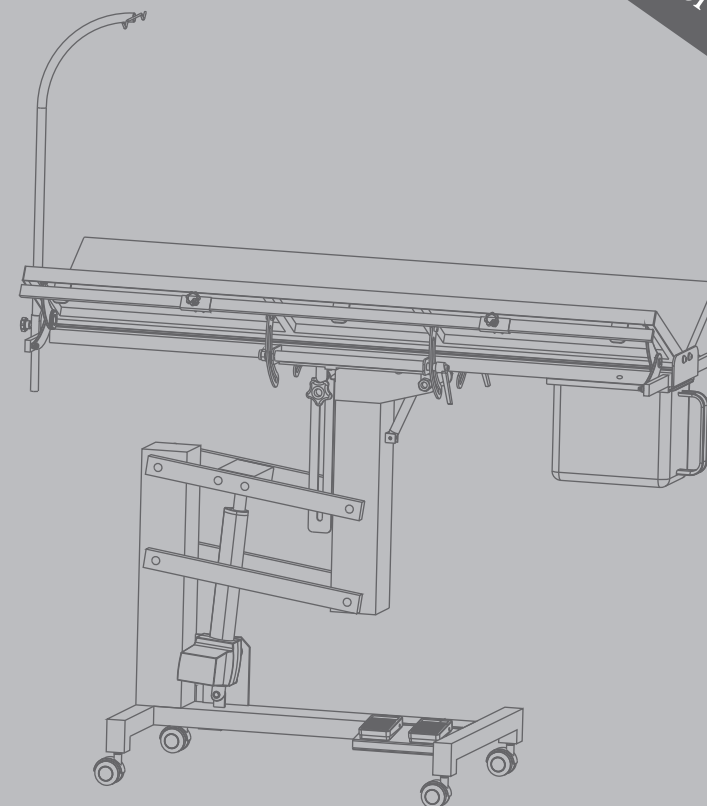
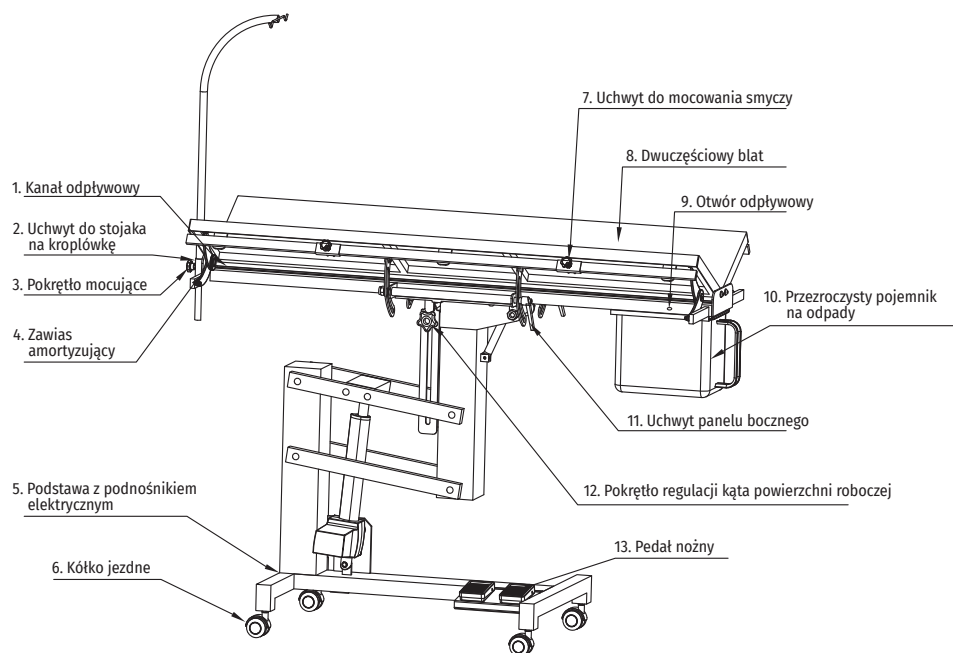
Figure 5

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